SPARE PART REPLACEMENT MANUAL

Ambu® aBox™ 2

For use by qualified technicians only.


Ambu

Contents

Page

English (Instructions for use)	1-13
Български (Инструкция за употреба).....	14-26
Čeština (návod k použití).....	27-37
Dansk (Brugsvejledning)	38-48
Deutsch (Bedienungsanleitung)	49-60
Ελληνικά (Οδηγίες χρήσης)	61-72
Español (Instrucciones de uso)	73-84
Eesti (Kasutusjuhend)	85-95
Suomi (Käyttöohjeet).....	96-106
Français (Mode d'emploi).....	107-117
Hrvatski (Upute za upotrebu).....	118-128
Magyar (Használati útmutató)	129-139
Italiano (Istruzioni per l'uso).....	140-150
日本語(使用説明).....	151-161
Lietuvių (naudojimo instrukcija)	162-173
Latviešu val. (Lietošanas instrukcija)	174-185
Nederlands (Gebruiksaanwijzing)	186-198
Norsk (Bruksanvisning)	199-210
Polski (Instrukcja użytkowania)	211-223
Português (Instruções de utilização).....	224-236
Română (Instrucțiuni de utilizare)	237-248
Slovensky (Návod na použitie).....	249-260
Slovenščina (Navodila za uporabo).....	261-272
Svenska (bruksanvisning).....	273-284
Türkçe (Kullanma Talimatı).....	285-296
中文(使用说明).....	297-307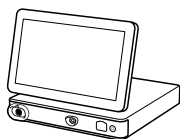

Content	Page
1. Important Information	4
1.1. Qualification Requirements for Personnel Performing Spare Part Replacement.....	4
1.2. Warnings and Cautions	4
1.3. Data Security	5
1.4. System Description.....	5
1.5. Spare Parts.....	6
1.6. Spare Part Replacement Process.....	7
2. How To Clean and Disinfect the Displaying Unit.....	7
3. Preparing for Spare Part Replacement.....	9
3.1. Tools Needed	9
4. How to Replace Spare Parts	9
4.1. Visualization Device Interface Board Replacement.....	10
5. Final Inspection and Test.....	13

1. Important Information

This is a Spare Part Replacement Manual for Ambu® aBox™ 2. It is applicable to aBox 2 with item number 505001000 only.

The following terms are used in this Spare Part Replacement Manual:

- “Displaying unit” refers to Ambu® aBox™ 2.
- “Visualization device” refers to compatible Ambu endoscopes and other compatible visualization devices that can be connected to and used with the displaying unit.

This manual may be updated without further notice. Contact your local Ambu representative or download the most recent version from ambu.com found on the download section on the product page.

Please observe that the warranty will be void, if parts of the displaying unit has been changed within the warranty period without the written consent from Ambu.

1.1. Qualification Requirements for Personnel Performing Spare Part Replacement

Do not start replacing any spare part of the displaying unit, unless you have read through this document and are qualified in the following areas:

- Knowledge, experience and acquaintance with electronic repair and safety testing according to IEC 60601-1 (Medical electrical equipment, Part 1: General requirements for basic safety and essential performance) and IEC 62353 (Medical electrical equipment – Recurrent test and test after repair of medical equipment).
- Knowledge of and experience with local regulations.
- Having read the Instruction for Use for the displaying unit (find the Instruction for Use in the sales package or download the current version from ambu.com).
- Knowledge of the environment where the displaying unit is installed and used.

1.2. Warnings and Cautions

WARNINGS

- To avoid risk of electric shock, always disconnect the power cable from the displaying unit before performing spare part replacement.
- To avoid contamination, always clean and disinfect the displaying unit before and after spare part replacement and wear gloves during handling.

CAUTIONS

- To avoid harming the electrical components, always use ESD protection when disassembling, replacing a spare part, and reassembling the displaying unit.
- To ensure the displaying unit functions optimally and to avoid compromising electrical safety, always perform the tests specified in section 5 after it has been reassembled. If any part of the tests fails after spare part replacement, do not use the displaying unit.
- To avoid malfunction of the equipment, only use spare parts supplied by Ambu. Do not modify the spare parts.
- Cleaning and disinfection wipes shall be moist, but not dripping to ensure no damage to internal electronics of the displaying unit.
- If using wipes containing hypochlorite or citric acid for cleaning, ensure that all residue is completely removed. Wipes containing hypochlorite or citric acid may affect the screen's antireflective coatings over time. You should limit the use of wipes containing hypochlorite or citric acid to required cases only.

1.3. Data Security

To avoid possible loss of data, it is recommended to export important files before performing any spare part replacement (please refer to the *Instruction for Use*).

1.4. System Description

The displaying unit is intended to display live imaging data from Ambu visualization devices.

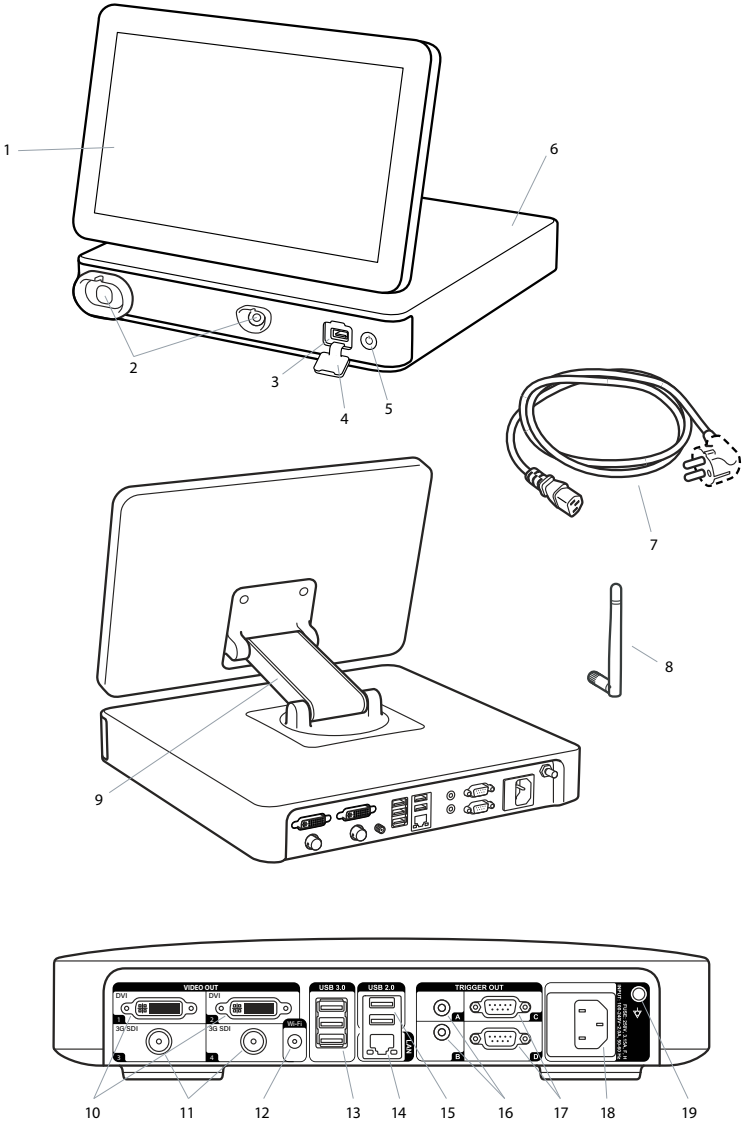
Figure 1 System illustration


No.	Part	Function
1	Touch Screen	Displays the image from the Ambu visualization device and the graphical user interface.
2	VDI port (connector port for specific Ambu visualization devices)	Connector port geometry and colour ensures correct connection with compatible visualization devices.
3	USB 3.0 port (front)	Enables connection of external USB flash drives.
4	USB cover	Protects the front USB port.
5	Power button	Turns the power ON or switches to STAND-BY mode.
6	Base	Contains the main unit.
7	Power cable	Connects the displaying unit to a power outlet.
8	Wi-Fi antenna	Connect Wi-Fi antenna to the displaying unit for improved Wi-Fi signal.
9	Positioning arm	Enables manual positioning of the touch screen. The screen can be adjusted horizontally and vertically as well as rotated.
10	Video output ports (2 x DVI-D)	Enable connection to external monitor or external medical imaging recorder.
11	Video output ports (2 x 3G-SDI)	
12	Wi-Fi antenna connector	Enables connection of Wi-Fi antenna.
13	USB 3.0 ports	Enable connection of external USB flash drives.
14	LAN port	Enables connection to ethernet.
15	USB 2.0 ports	Enable connection of external USB flash drives.
16	Trigger output ports (2 x 3.5 mm jack)	Enable connection to an external medical imaging recorder to transfer trigger signals.
17	Trigger output ports (2 x D-SUB9)	
18	Power inlet	Enables connection to power cable.
19	Connector for potential equalization cable	Used for bonding of electrical products to eliminate potential differences between conductive parts.

1.5. Spare Parts

The spare parts may not be available in all countries. Please contact your local Ambu representative.

CAUTIONS

- To avoid malfunction of the equipment, only use spare parts supplied by Ambu. Do not modify the spare parts.

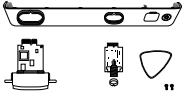Spare Part	Description	Item numbers
	Ambu® aBox™ 2 – Visualization device interface kit – Grey-Empty-Green Contains: One grey and one green visualization device interface board (VDI), a front cover with a colour ring (grey and green), a plectrum tool, and two screws for the VDI.	505000530
	Power cables – J (CH), K (DK), I (AUS/NZ)	505000520
	Power cable – B (US, JP)	505000521
	Power cables – G (UK), E/F (EU, not DK, CH)	505000522

1.6. Spare Part Replacement Process

This section specifies a spare part replacement process which is in compliance with IEC 62353.

Read these Instructions for Use carefully before performing any replacements on the Ambu® aBox™ 2.

The technician and personnel being responsible for the replacement process should fulfil the requirements outlined in section 1.1.

Follow the process in Figure 2 to replace spare parts in the displaying unit.

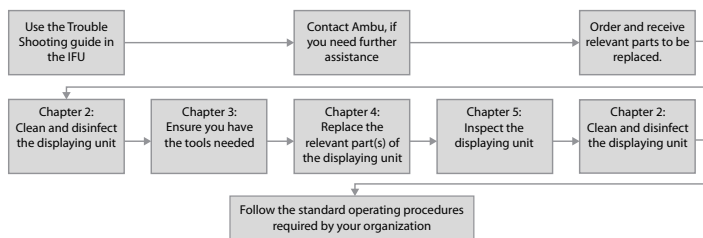
Figure 2 Spare part replacement flow

2. How To Clean and Disinfect the Displaying Unit

CAUTIONS

- Cleaning and disinfection wipes shall be moist, but not dripping to ensure no damage to internal electronics of the displaying unit.
- If using wipes containing hypochlorite or citric acid, ensure that all residue is completely removed. Wipes containing hypochlorite or citric acid may affect the screen's antireflective coatings over time. You should limit the use of wipes containing hypochlorite or citric acid to required cases only.

The displaying unit is a reusable medical device. According to the Spaulding classification, the displaying unit is a non-critical device.

The displaying unit should be cleaned and disinfected before and after repair and replacement procedures following one of the procedures below.

Any deviation from the instructions should be properly evaluated for effectiveness and potential adverse consequences by the person responsible for cleaning and disinfection, to ensure that the device continues to fulfil its intended purpose. Cleaning procedures should

begin as soon as possible following use. Excessive soil should be removed on accessible areas of the device, excluding electrical ports.

Limitations: The displaying unit is not compatible with ultrasonic or automatic cleaners and should not be immersed.

Procedure 1 – Cleaning and disinfection with hypochlorite

Hypochlorite-based wipes approved for disinfection of medical devices, e.g., Sani-Cloth® Bleach from PDI, should be used in accordance with the wipe manufacturer's instructions.

Cleaning: Use a wipe to remove heavy soil. All blood and other body fluids must be thoroughly cleaned from surfaces and objects. Inspect the displaying unit for cleanliness, function and integrity before disinfection by germicidal wipe. If visible soil remains, reclean the displaying unit.

Disinfecting:

1. For heavily soiled surfaces, use a wipe to pre-clean the displaying unit prior to disinfecting.
2. Unfold a clean wipe and thoroughly wet the surface of the displaying unit.
3. The treated surfaces must remain visibly wet for a full four (4) minutes (or the time recommended by the manufacturer of the disinfectant, at least 4 minutes). Use additional wipes if needed to assure continuous 4 minutes wet contact time.
4. Let the displaying unit air dry.

Procedure 2 – Cleaning and disinfection with quaternary ammonium compounds

Wipes containing a mixture of quaternary ammonium compounds and isopropyl alcohol approved for disinfection of medical devices, e.g., Super Sani-Cloth® from PDI, should be used in accordance with the wipe manufacturer's instructions.

Cleaning: Use a wipe to remove heavy soil. All blood and other body fluids must be thoroughly cleaned from surfaces and objects. Inspect the displaying unit for cleanliness, function, and integrity before disinfection by germicidal wipe. If visible soil remains, reclean the displaying unit.

Disinfecting:

1. For heavily soiled surfaces, use a wipe to pre-clean the displaying unit prior to disinfecting.
2. Unfold a clean wipe and thoroughly wet the surface of the displaying unit.
3. The treated surfaces must remain visibly wet for a full two (2) minutes (or the time recommended by the manufacturer of the disinfectant, at least 2 minutes). Use additional wipes if needed to assure continuous 2 minutes wet contact time.
4. Let the displaying unit air dry.

Procedure 3 – Enzymatic detergent cleaning and alcohol disinfection

Cleaning:

1. Prepare a cleaning solution using a standard enzymatic detergent prepared per manufacturers recommendations. Recommended detergent: enzymatic, mild pH: 7 – 9, low foaming (Enzol or equivalent).
2. Soak a sterile gauze in the enzymatic solution and make sure that the gauze is moist and not dripping.
3. Thoroughly clean the button, external side of rubber covers, screen, external casing of the monitor with the moist gauze. Avoid getting the displaying unit wet to prevent damaging internal electronic components.
4. Wait for 10 minutes (or the time recommended by the manufacturer of the detergent) to allow the enzymes to activate.

5. Wipe the displaying unit clean using sterile gauze that has been moistened with RO/DI water. Ensure all traces of the detergent are removed.
6. Repeat steps 1 to 5.

Disinfecting: Wipe the surfaces of the displaying unit for approximately 15 minutes using a piece of sterile gauze moistened with the alcohol mixture indicated below (approximately once every 2 minutes). Follow safety procedures for the handling of isopropyl. The gauze should be moist and not dripping since liquid can affect the electronics inside the displaying unit. Pay close attention to the button, external side of rubber covers, screen, external casing and slots and gaps on the displaying unit. Use a sterile cotton swab for these areas. Solution: Isopropyl (alcohol) 95 %; Concentration: 70 – 80 %; Preparation: 80 cc of 95 % Isopropyl (alcohol) added to 20 cc of purified water (PURW). Alternatively, use EPA-registered hospital disinfection wipes containing at least 70 % isopropyl. Safety precautions and directions of use of the manufacturer must be followed.

NOTE: After cleaning and disinfection, the displaying unit must be submitted to the pre-check procedure in chapter 5.

3. Preparing for Spare Part Replacement

3.1. Tools Needed

- Plectrum tool
- Screwdriver (Phillips #0) – Not supplied
- Compatible (and functioning) Visualization Device – Not supplied

The plectrum tool is provided with the spare parts kits (see section 1.5 Spare Parts).

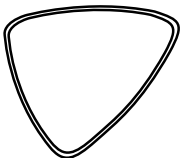
Figure 3 Plectrum tool

4. How to Replace Spare Parts

This section covers the replacement of the Visualization Device Interface board (VDI board).




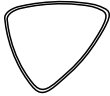
WARNINGS

- To avoid risk of electric shock, always disconnect the power cable from the displaying unit before performing spare part replacement.

CAUTIONS

- To avoid harming the electrical components, always use ESD protection when disassembling, replacing a spare part, and reassembling the displaying unit.
- To avoid malfunction of the equipment, only use spare parts supplied by Ambu. Do not modify the spare parts.

4.1. Visualization Device Interface Board Replacement

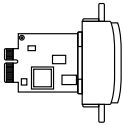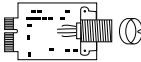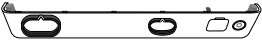Ambu® aBox™ 2 Visualization Device Interface Kit – Grey-Empty-Green	505000530
Grey Visualization Device Interface board (VDI board)	
Green Visualization Device Interface board (VDI board)	
Front cover, incl. colour ring (grey) and USB cover	
Screws for VDI board (x2)	
Plectrum tool	

Additional Tools (not supplied)

Screwdriver (Phillips #0)

IMPORTANT:

Ensure all cables and equipment are disconnected from the displaying unit before initiating the replacement procedure!

The pictures are for illustration purposes. The number, color and position of front connectors may differ. Please make sure to install the exchanged parts at the original positions.

a) Detach the front cover from the displaying unit

Locate the groove on one of the sides of the front cover where a snap feature is located on the inside. Press inwards with the plectrum tool and a finger to detach the front cover from one side.


NOTE: Be careful not to press on the top or bottom of the displaying unit, as it will complicate removal of the front cover.


- b) Completely remove the front cover by pulling it carefully from the displaying unit

- c) Remove the 2 screws holding each VDI board using a Phillips screwdriver

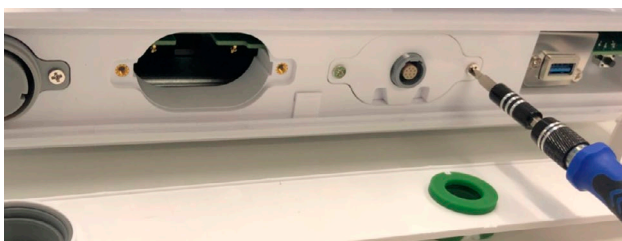
- d) Pull out the VDI boards from the displaying unit and dispose them according to local guidelines for disposal of electronic waste

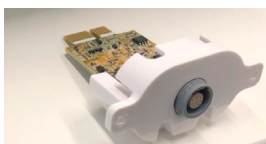
- e) **Carefully insert the new VDI boards into their original positions**
 Ensure that the VDI boards are oriented correctly as shown on the picture below.
NOTE: Insert the VDI boards straight and horizontally into the original main board connector positions to ensure correct connection of the VDI board to the displaying unit.

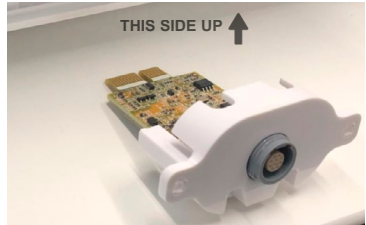- f) **Carefully fasten the VDI boards with the supplied screws using the Phillips screwdriver into their original positions**

- g) **Attach the new front cover supplied with the replacement kit**
 Start by attaching from one end. Ensure that the snap features on each end is placed inside the housing of the displaying unit. The front cover is pressed in place until all top and bottom snap features click in place.
NOTE: Ensure that the colour rings are fitted correctly in the front cover and that colours and arrows on the front cover and the displaying unit match.
NOTE: Ensure the USB cover flap is inserted underneath the USB port.

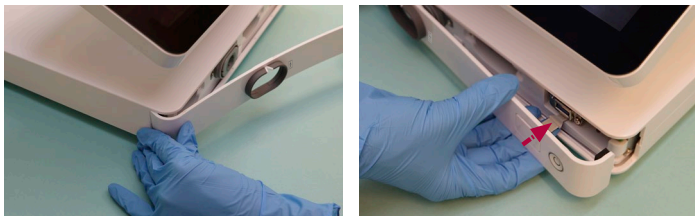5. Final Inspection and Test

CAUTIONS

- To ensure the displaying unit functions optimally and to avoid compromising electrical safety, always perform the tests specified in section 5 after it has been reassembled. If any part of the tests fails after spare part replacement, do not use the displaying unit.

Perform the following inspections and tests after assembly prior to use:





a) Examine displaying unit for visual damage

Closely examine the displaying unit and other parts for any damage . Do not use the displaying unit if it is damaged in any way.

b) Visual inspection with a functioning scope

Connect the displaying unit to a suitable power outlet via the power cable . Connect a functioning visualization device to the displaying unit by plugging it into the visualization device connector (VDC) port . Ensure that the connector colours are matching, and the arrows are aligned. Turn on the displaying unit on the power button.

Verify that a live video image appears on the screen by pointing the distal end of the Ambu visualization device towards an object, e.g. the palm of your hand, when the Live View  tab is selected in the toolbar to the left .

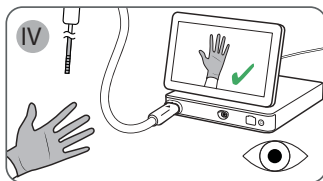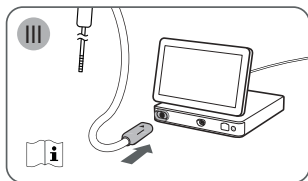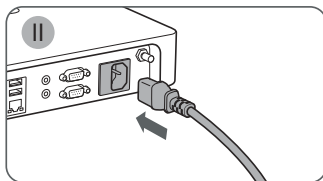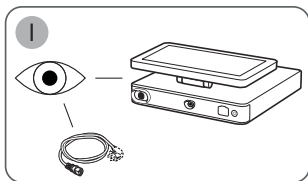For other relevant testing of the displaying unit, refer to the requirements of IEC 62353 or other relevant safety tests according to local guidelines.

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Ambu is a registered trademark and aScope and aBox are trademarks of Ambu A/S.
Ambu A/S
Ambu Ltd